
VOYAGE QUICK-RELEASE SEATING SYSTEM TETHER




Please read these instructions carefully before use. Failure to understand and follow instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.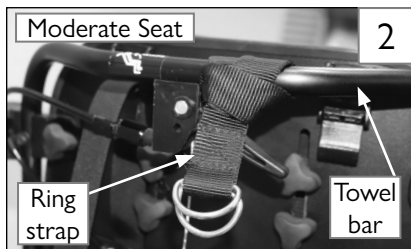

Parts included:

1. Ring strap
2. Quick-release tether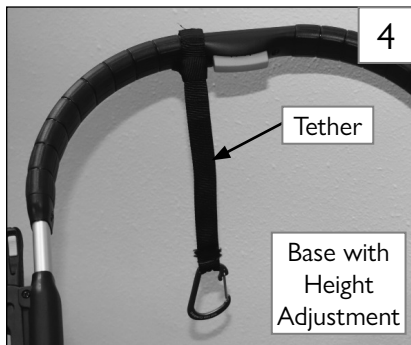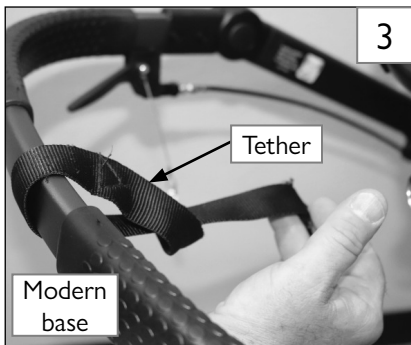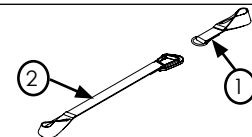



A. ATTACH RING STRAP TO SEATING SYSTEM TOWEL BAR 1 2

Note: If system is fitted with transit strap (Fig. 7, Fig. 8), skip step and proceed to step B.

1. From the top of the towel bar, loop strap around towel bar and pull ring through loop until the strap is snug around the bar as shown (Fig 1). Ensure strap is relatively centered on the towel bar and does not interfere with recline levers.

B. ATTACH QUICK-RELEASE TETHER TO PUSH HANDLE OF BASE FRAME 3 4

1. From the top of the base frame, loop strap around push handle and pull carabiner through loop until the strap is snug around the push handle, as shown (Fig 3). Ensure strap is relatively centered on the push handle and does not interfere with height adjustment of the push handle (Fig 4).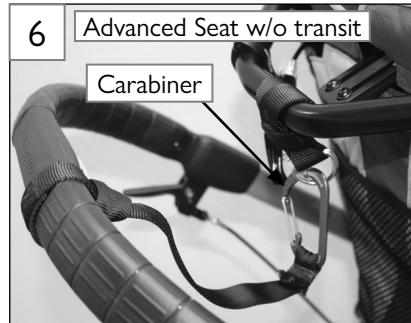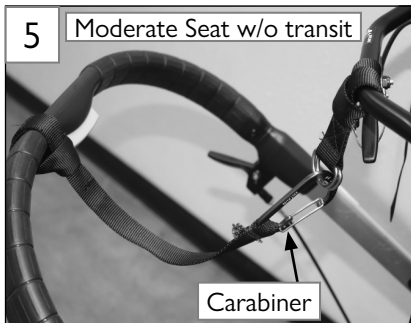



NOTE: Once steps A and B are completed, they will not need to be repeated in order to use the seating system tether. Steps C and D must be repeated each time you remove and reattach the seating system.

C. ATTACH SEATING SYSTEM TO BASE

1. Ensure seating system is properly installed on stroller base before proceeding to step D.

NOTE: Refer to the owner's manual for seat installation and removal instructions.

D. CONNECT TETHER STRAP TO RING STRAP 5 6 7 8

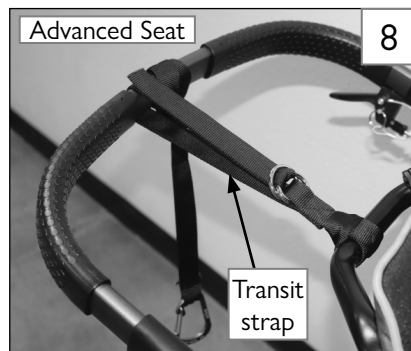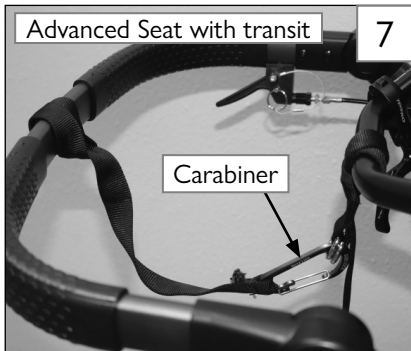1. Connect the tether strap to the ring strap or existing transit D-rings using carabiner, as shown (Fig 3 - Fig 5). Ensure carabiner gate is fully closed.
 2. System is now ready to use.
- Note:** Carabiner must be unclipped from ring strap before removing seat from base.

⚠ WARNING!

1. When chair is in the forward-facing configuration, seating tether is ALWAYS required.
2. The seating tether IS NOT a replacement for the factory-installed transit strap. For all transit situations you must use the transit strap provided with your device (as shown in Fig 8). It secures the seating shell to the rear frame. Refer to the owner's manual for instructions for use.

⚠ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Zippie wheelchair. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

See page 8 for contact information or visit www.SunriseMedical.com

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© 2022 Sunrise Medical (US) LLC
Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S